

# POB-MA LOADING SHEET

Protocol for P2 upgraded preloaded products:  
877PA(Y), 877PEY

**3!** POB-MA  
FLEX | HYDROPHOBIC PRELOADED

The loading sequence is indicated on the injector body, with the numbers 1-4


1.

**Fully introduce the OVD cannula** into the small aperture on the right side of the injector, indicated with '1'.  
Make sure that it is completely inserted into the blue funnel.


**Inject OVD in a slow and controlled manner** until the IOL haptics are covered.

The recommended **OVD amount is 0.2-0.25 ml**.

**Check the lens position** in the cartridge, make sure it is located within the cartridge rims.

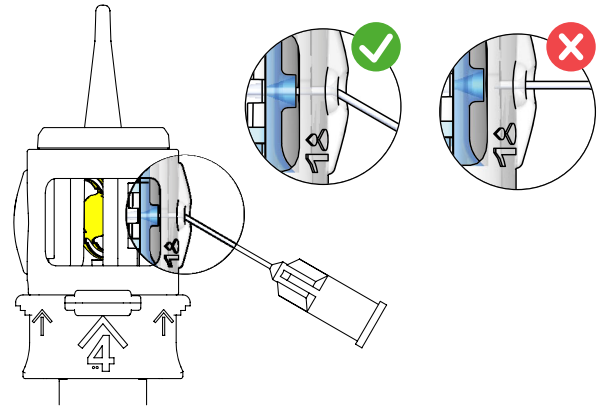


2.

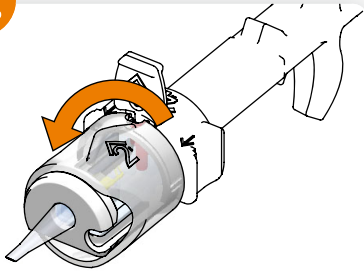


**Rotate the transparent ring by 90 degrees**, indicated by '2' to close the cartridge, until you hear a „click“.

3.

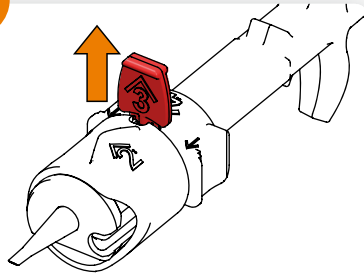


**Remove the red stopper** indicated with '3'.

4.

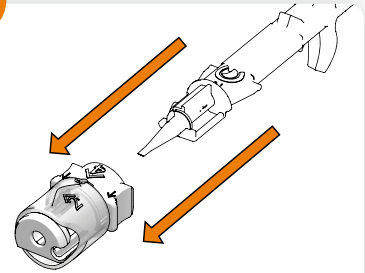

**Pull off the rotational ring adapter** indicated with '4'.


5.

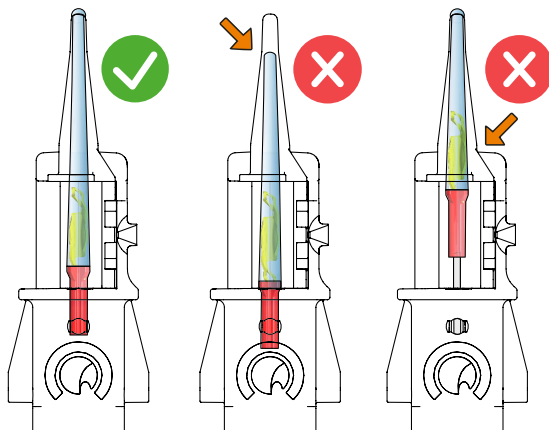

**To activate the coating in the nozzle**, advance the plunger a few millimeters until the **OVD reaches the end of the nozzle tip**, and **wait 30 seconds**.

**Make sure that the IOL remains in the cartridge**, and **do not push the IOL into the nozzle** while waiting. If necessary, fill in some OVD through the nozzle.

6.

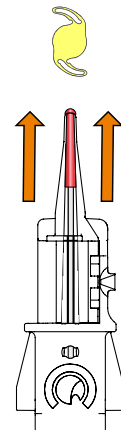

**Now you are ready for injection**. Push the plunger forward continuously in a **slow, controlled manner** and **do not pause** until the IOL exits the nozzle.