


3! | POB-MA

FLEX | HYDROPHOBIC PRELOADED



EASY, SAFE AND
PREDICTABLE
INJECTION SYSTEM

MEDICENTUR

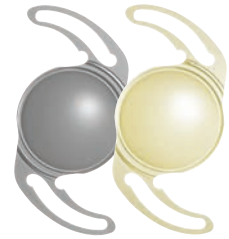

1st STEP Fill visco in the hole until the optic of the lens is covered (preferably HPMC)

2nd STEP Rotate the transparent ring to close the cartridge

3rd STEP Keep the injector at this state for a minimum of 3 mins

3 min

4th STEP Pull out the red stopper

5th STEP Remove the ring-adapter from the injector

INJECTION Push the plunger continuously and do not pause until the optic exits the cartridge tip

Type Single-piece Aspheric Hydrophobic Acrylic IOL

Material Hydrophobic Acrylic with UV absorber /
 + optional blue light filter

Diameters 6.0 mm (optic) / 13.0 mm (overall)

Angulation 0° – posterior vaulting

Power range -10.0D → +9.0D · (increment: 1.0 D)
 +10.0D → +30.0D · (increment: 0.5 D)
 +31.0D → +35.0D · (increment: 1.0 D)

A-Constant 118.9 (SRK/T)

* Patent pending

EA – IMCPOBMAENG 07 201812